
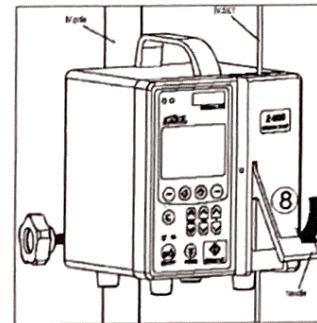
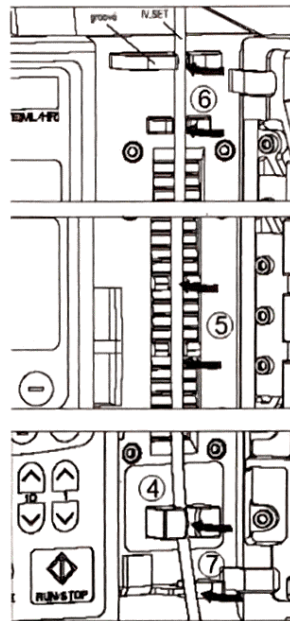


Quick User Guide

Z-800 Infusion Pump System

TURN ON THE PUMP
By pressing the ON/OFF button for 2 seconds

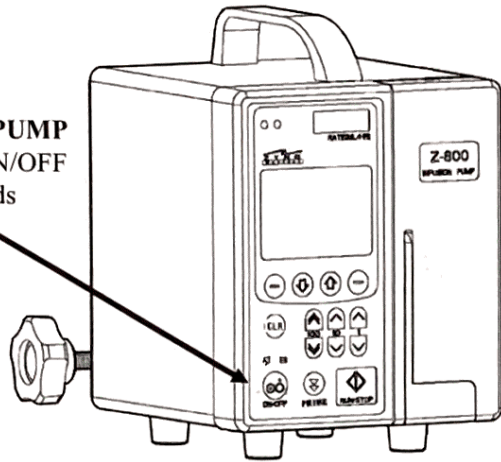

INSTALL IV SET

1. Prime IV set.
2. Close roller clamp
3. Open pump door.
4. Push tubing into the gap of Air-in-Line Sensor.
5. Push tubing into the gap of the tubing guides on the pumping fingers.
6. Push tubing into the gap of upper tubing guides.
7. Align tubing on top of the gap of Free Flow Clamp (Do not force tubing into the Free Flow Clamp). The Z-800 Pump will load the tubing into the Free Flow Clamp automatically when the pump door is closed.
8. Close pump door by pushing down the pump door handle.
9. Open roller clamp. Confirm NO FLOW in drip chamber

PRIMARY INFUSION SETUP


SELECT PARAMETER
using UP/DN
Arrow buttons



MODIFY PARAMETERS
using data entry
buttons




ACCESS SECONDARY INFUSION SETUP

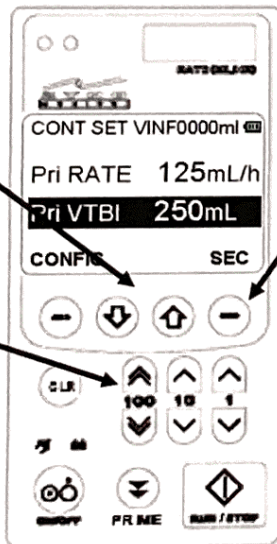

SECONDARY INFUSION SETUP

ACCESS PRIMARY INFUSION SETUP

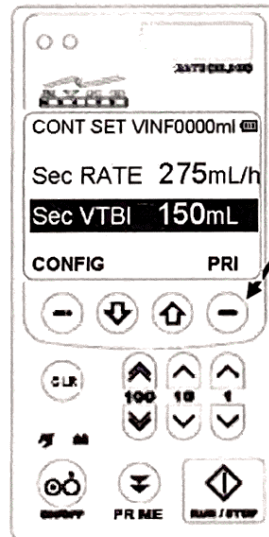

START OR STOP AN INFUSION

START OR STOP AN INFUSION
Using
RUN/STOP
button

